

eakin Cohesive[®] paste

Alcohol-free, no-sting adhesive paste for use in ostomy, wound & fistula care

Recommended
by **97%**
of nurses¹

3 Great reasons to use eakin Cohesive[®] paste:

1 Skin friendly filler containing no alcohol in it's non-sting formulation making it ideal for sensitive, broken or delicate skins.

In a recent survey, 93% rated it as good or excellent for skin friendliness.¹

80% found improvement in skin quality after using eakin Cohesive[®] paste.¹

“ Easy to apply and it causes no pain on skin lesions
(Stomatherapy Nurse, Paris, France)

Comfortable, skin friendly paste ”
(Stomatherapy Nurse, Northern France)

2 Levels the skin surface and helps prevent leaks by forming an effective barrier and level skin surface around the stoma, fistula or wound.

86% of nurses said there were no leaks while using eakin Cohesive[®] paste.¹

3 Fills in crevices, skin folds and scars helping to increase pouch wear time.

85% felt more secure when using eakin Cohesive[®] paste.¹

65% were able to leave their pouch on for at least 48 hours (and even up to 96 hours in some cases).¹

“ Better wear time for a complex pouch ”
(Stomatherapy Nurse, Paris, France)

Plus it's:

- ✓ An effective skin protector
90% rated it good or excellent at protecting the skin¹
- ✓ Very easy to use
93% of nurses felt eakin Cohesive[®] paste was easy to use
93% felt it was easy to remove
73% found it left very little or no residue

“ Excellent product, easy to use ”
(Stomatherapy Nurse, Paris, France)



A SMALL PART
OF A BIGGER LIFE.



Choose eakin Cohesive® paste for use in Stoma Management

Problem:

- Lady with a urostomy had been experiencing leaks at 3 o'clock where a crease leads to the midline scar.
- Contacted nurse for advice.

Nurse Review:

- Nurse recommends use of eakin Cohesive® paste to fill in gaps and creases around the stoma.

Outcome:

- Successful seal achieved with soft convex pouch, eakin Cohesive Slims® and eakin Cohesive® paste to fill the crease.
- The lady prefers eakin Cohesive® paste to other pastes that she had tried as they left a residue which was difficult to remove.



Choose eakin Cohesive® paste for use in Wound & Fistula Management

Problem:

- This lady had emergency surgery for a bowel perforation. She subsequently developed enterocutaneous fistulas.

Nurse Review:

- Several pouches were trialled before commencing on eakin Wound Pouches™.
- She was still having daily leaks causing painful skin excoriation.
- Nurse recommended eakin Cohesive® paste to deep creases left and right of lower fistula and cut separate holes in the pouch for the fistula.

Outcome:

- The pouch remained intact for 72 hours which resulted in an improvement in the skin condition.



Choose eakin Cohesive® paste for use around a tube

Patient history:

- 59 year old male with a tumour down the oesophagus, required a jejunostomy with the implantation of a port.
- Food is semi-liquid, taken naturally and enteral nutrition.

Problem:

- 3 weeks later Mr F is in a lot of pain and has skin damage so the nurse creates a care procedure using saline solution and a Biosorb pad.

Nurse Review:

- 2 days later Mr F is still experiencing a lot of pain and skin damage so the nurse creates a new care procedure using a large hydrocolloid which allows a catheter to go through.
- The nurse creates a barrier with eakin Cohesive® paste and a secondary plaster on top.

Outcome:

- After 5 days, there is no more pain and skin around the stoma is improved so the same care procedure is continued.
- eakin Cohesive® paste was chosen as it's alcohol-free, so no pain when it is put on the skin and it levels the skin surface by filling in skin wrinkles.

