

# eakin **Cohesive**<sup>®</sup> stomawrap with hernias and larger stomas



A SMALL PART  
OF A BIGGER LIFE.

## Background

- 76 year old gentleman.
- 2014: End ileostomy formed due to Colorectal Cancer.

## Problems Experienced

- Developed large parastomal hernia however deemed unsuitable for surgical repair due to multiple co-morbidities.
- Stoma prolapsed and measured 80mm.
- Experiencing regular pouch leakages.
- Trial of 1 piece Easifold Maxi which facilitated 80mm aperture combined with a regular seal.
- Seals thinning too much when stretched to 80mm.

## Nurse Advice

eakin **Cohesive**<sup>®</sup> stomawrap was recommended due to it's unique shape however a single seal was also thinning when stretched.

Secure seal achieved by utilising 2 x eakin **Cohesive**<sup>®</sup> stomawrap seals, placing one of top of each other and stretching to 80mm allows for the seal to fully stretch around the stoma with no thinness.

## Solution with eakin **Cohesive**<sup>®</sup> stomawrap

Solution has been used for a year and has resulted in great seal security and skin protection.

**Skin:** Rarely experiences a leak meaning peristomal skin remains healthy.

**Stoma Management:** Confident in applying the eakin **Cohesive**<sup>®</sup> stomawrap doubled up to protect his peristomal skin.

**Bigger Life:** Average weartime with the seal is 3 days allowing the gentleman to get on with his life without worrying about leaks.

