
EN **eakin freeseal**[®] are moisture absorbing, skin protecting seals which protect peristomal or periwound skin from harmful attack by aggressive body fluids. High tackiness on both sides ensures that any subsequent appliance is held securely in position on the body.

Indications for use

- An ostomy seal.
- A sealing agent around fistula and wounds.
- An external packing agent on uneven skin.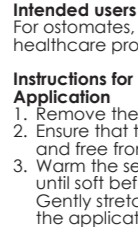

EN Warm the seal in your hands for 20 – 30 seconds until soft before removing the protective discs. Gently stretch the **eakin freeseal**[®] if necessary to fit the application site. It may also be broken or cut and re-joined if necessary to fit around difficult sites.

4. Apply to the skin with gentle pressure and ensure the **eakin freeseal**[®] is securely attached. Mould the seal towards the base of the stoma or the edge of the wound.
5. Ensure a snug fit with no gaps around the base of the stoma or edge of the wound to provide maximum skin protection.
6. Place the intended appliance directly onto the surface of the **eakin freeseal**[®]. Hold in place with gentle pressure until securely attached to the **eakin freeseal**[®] and the surrounding skin.

Removal

Gently remove the appliance from the body. Remove the **eakin freeseal**[®] by carefully peeling from the outer edge. For ease of use, the majority of the seal should come away in one piece.

Dispose

Dispose of waste products in accordance with local guidelines.

Warnings and cautions

Consult a healthcare professional if you have any skin irritation, redness or discomfort in the area of product application prior to use. If during use you experience any of these symptoms, discontinue use and consult a healthcare professional. Do not apply to broken skin or open wounds.

Other information

Store flat in the carton at room temperature between 10° and 30° Celsius. Do not refrigerate.

eakin[®] accepts no liability for any injury or other loss that may arise if this product is used in a manner contrary to **eakin**[®]'s current recommendations.

Any serious incident encountered in relation to the use of this device should be reported to the manufacturer and the competent authority in the country of use. A serious incident is one involving serious deterioration in health or patient death.

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